


## GUIDER MANUAL

- Intended use

The one piece system needle guider with bracket is safer, adds efficiency and accuracy to ultrasound-guided procedures. Its easy and simple way of using, promotes reduced technique variability, which leads in a shorter learning curve and reduced procedure time



1. Apply the guider  
Please note to put the guider  
at the side mark  
(different from probe to probe)  
and the closing at opposite side.

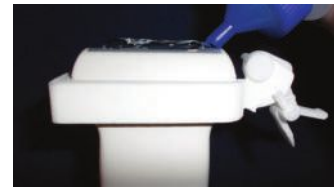

2. Apply gel



3. Open package of the sterile  
ECO-cover Kit.  
Insert the probe with guider into  
the cover by using sterile gloves and  
proper sterile technique



4. Pull the cover over  
The probe



5. Place the guider with the probe as lateral as possible to the opposite side of the guider



6, Secure the cover with enclosed bands. Use one of the bands between needle guide and probe. Please note to give guider enough place to move. Use the second rubber band around probe



7. Press to open Guider for any size of needle Gauge

Lay needle into guide holder-  
Do NOT slide needle into holder  
-Risk for puncturing the cover



8. Pull needle backwards to Zero-position to start procedure. During scanning, you can rotate the wheel for selected depth.

Make sure the tip of the needle stays in zero position for NO RISK for puncturing the Cover.



BE/CA01/2-12946-00001-CLI

Manufactured by  
**4ech**   
Member of Mepy



A Mepy Benelux Division  
Zeedijk 130, 8400 Oostende  
Belgium  
[www.4echo.com](http://www.4echo.com)